




Connected to streaming device

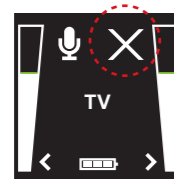

No streaming device found

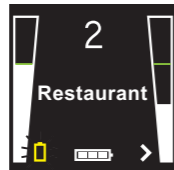


Low battery on left hearing instrument

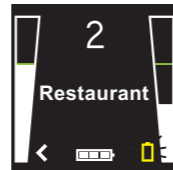


Low battery on right hearing instrument


For more details on the Remote Control 2, refer to the user guide.

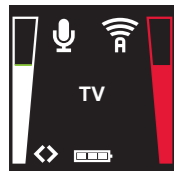

Left and right hearing instrument volume selected

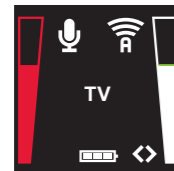

Left and right streaming volume selected

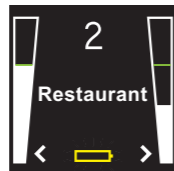

Low battery on Remote Control 2



Phone program activated



GETTING STARTED (QUICK GUIDE)

Remote Control 2 for Model KS562-DRW hearing instruments



Every Kirkland Signature product is guaranteed to meet or exceed the quality standards of the leading national brands. If you're not completely satisfied, your money will be refunded.

Distributed by:
Costco Wholesale Corporation
P.O. Box 34535
Seattle, WA 98124-1535
USA
1-800-774-2678
www.costco.com

Find out more at Costco.com

400245234-US Rev.A 2013.07

13V1193b

A GETTING STARTED

Charging the Battery
Before using your Remote Control 2, you need to fully charge the rechargeable battery. To charge it, plug in the charging cable as shown to the right. Then plug the other end into a power outlet. During charging, the bars in the battery meter will be showing one, two and three bars continuously. When fully charged all three bars will be shown continuously.



NOTE! Check with your hearing healthcare professional to determine if your instruments have been pre-paired. Instruments that have been pre-paired do not need to be paired again.

Turning your Remote Control 2 ON and OFF
To turn your Remote Control 2 ON, slide the switch on the side of the Remote Control 2 down to the ON position.

A white "I" icon will be visible on the base of the switch and the display will turn on.

To turn your Remote Control OFF, slide the ON/OFF switch up to the OFF position.



Do not charge the Remote Control 2 while it is on your body.

B PAIRING YOUR REMOTE CONTROL

Make sure the Remote Control 2 is fully charged and is turned ON.



1. Open the battery doors on both hearing instruments
2. Press the pairing button on the Remote Control 2 to activate the pairing mode, which will be active for 20 seconds
3. While in pairing mode, close the battery doors on both hearing instruments




4. The Remote Control 2 will search for the hearing instruments.

5. A successful pairing will be indicated by an audible melody played in both hearing instruments, and shown in the Remote Control 2 display with active volume bars and program "1". You are now ready to use the Remote Control 2!

DISPLAY SCREENS

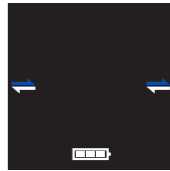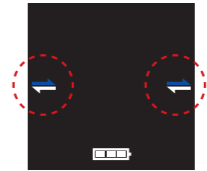


Searching for both hearing instruments

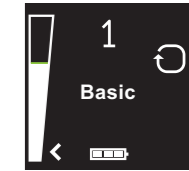

Connected to left hearing instrument and searching for right hearing instrument



Program 1 in left and right hearing instrument

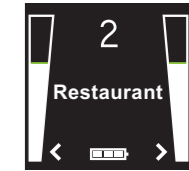

Program 2 in left and right hearing instrument